

VibraJect[®]

by A.TITAN

- Vibration to reduce the pain of routine injections
- Ergonomic design does not impede procedure
- Save time by eliminating block injections
- Treat multiple quadrants in a single visit
- Compatible with all syringe brands
- No need for topical anesthesia
- Less painful palatal injections
- Cordless / Battery-operated
- Patented technology
- Lifetime Guarantee
- Made in U.S.A.



Achieve
PAINLESS
Injections



VibraJect[®]

by A.TITAN

VibraJect System (#VJ2002)

Includes:

- (1) VibraJect Motor
- (3) Syringe Clips
- (6) Silicone Sleeves
- (4) Disposable Batteries

VibraJect[®] R3

by A.TITAN

VibraJect R3 Complete Kit (#VJ2002R3)

Includes:

- (1) A.Titan harmonically-balanced syringe
- (1) VibraJect Motor
- (3) Syringe Clips
- (6) Silicone Sleeves
- (2) Rechargeable Batteries
- (1) Battery Charger

Learn more at ATitan.com/VibraJect



VibraJect[®]

by A.TITAN



"The VibraJect offers an 'ouchless injection' solution for anesthetizing patients quickly, in a more comfortable manner."

Dr. Howard S. Glazer, Fort Lee, New Jersey

Buccal Furca PDL technique *by Dr. Howard Glazer*

Step 1: Attach the **VibraJect** to the barrel of a syringe.

Step 2: Use 30 gauge extra short needle.

Step 3: Turn on the **VibraJect**, approach the buccal sulcus of the tooth and slowly drip anesthetic into the sulcus.

Step 4: Continue to slowly advance and drip the anesthetic fluid until resistance of the periodontal ligament is felt.


Step 5: Continuing in a slow fluid movement penetrate the ligament and place a few more drops of anesthetic.

Step 6: The same procedure is repeated for the lingual of the tooth, always maintaining a slow drip and forward progression.

Step 7: Turn off the **VibraJect** and recap the needle.

**If necessary, this procedure may be repeated at the mesio-buccal, disto-buccal, mesio-lingual and disto-lingual of the tooth.*

***Anesthesia is rapid and profound and will generally last 40-60 minutes, usually with less than 1/2 carpule of anesthesia.*

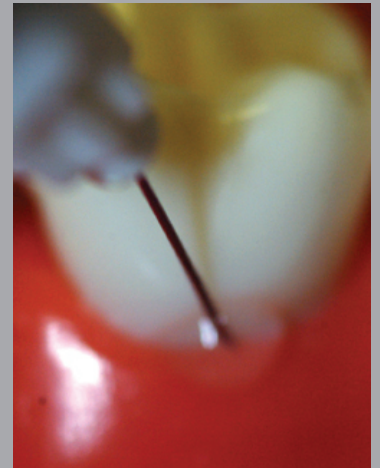

Learn More at **VibraJect.com**



A.Titan Instruments, Inc. 10 Centre Drive Orchard Park, NY 14127
1-877-284-8261 • 716-667-9211 • www.ATitan.com